




# MEDI-FLO<sup>®</sup> MANUAL PUMP FILLER

## Why MEDI-FLO<sup>®</sup> Manual Pump Filler?

The MEDI-FLO<sup>®</sup> Manual Pump Filler is an easy and convenient way to fill elastomeric pumps. Our Manual Pump Filler is compatible with all 60 mL luer lock syringes and elastomeric pumps on the market. Fill your syringe with medication, load into the filler, attach the elastomeric pump to the syringe, and pull the handle down to inject the medication into the pump. The ergonomic design takes the pressure off your hands and wrists. Suction cups and rubber feet help to secure the filler to your workstation.

## Instructions for Use

1. Ensure filler is secured to your workspace.
2. Push back handle and open syringe guard.
3. After filling 60ml luer lock syringe with drug or diluent, load into hand filler by lining up the plunger flange and the barrel flange into their respective slots on the filler with the gradations facing up.
4. Close the syringe guard to hold the syringe in place while filling.
5. Attach the elastomeric pump to the end of the luer lock syringe.
6. Pull down on the handle to inject medication or diluent into the elastomeric pump.
7. Detach the elastomeric pump from the syringe.

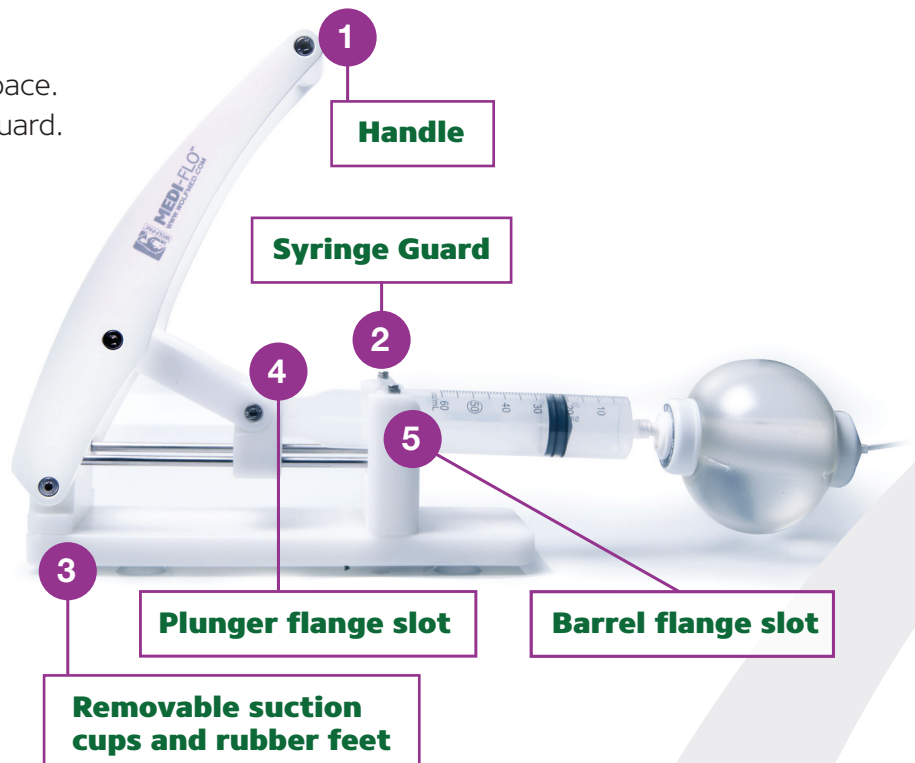

Item	Description	Quantity
MDH01	MEDI-FLO <sup>®</sup> Manual Pump Filler	1/ea

**IN STOCK & READY TO SHIP**

(800) 335-9653 | [www.wolfmed.com](http://www.wolfmed.com)

*Delivering Efficiency to Healthcare<sup>®</sup>*

*Distributed by: Wolf Medical Supply, An NDC Company  
13951 NW 8th Street | Sunrise, FL 33325  
[www.wolfmed.com](http://www.wolfmed.com) | 800.335.9653*